
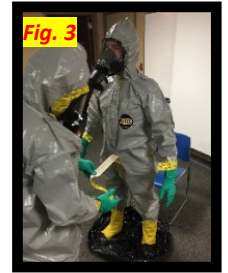


# Doffing of Level C (EMS Exposure Kit)




## Step 1: After a complete decontamination

- Remove tape from mask (Fig.1)
- Dispose of items into lined trash bin (Fig.2)
- Remove tape from front flap (Fig.3)

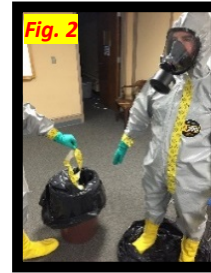


## Step 2: Have an assistant help remove the suit from the entrant. (Entrant should stand in

- Assistant should be in appropriate PPE/Assistant should only touch the outside of the suit
- Entrant or assistant should open flap and unzip the suit (Fig.4), then assistant helps get arms out of sleeves.(Fig.5)
- Goal is to remove suit without excessive “shaking”. Assistant “peels” the suit off the entrant (Fig.6, 7, 8)
- Entrant continues the removal of suit into the garbage bag. Entrant only touches inside of suit (Fig.9)

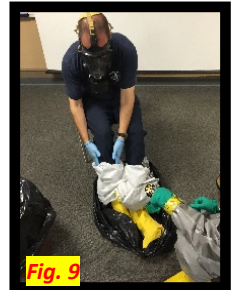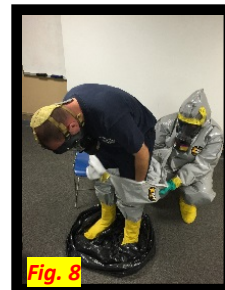
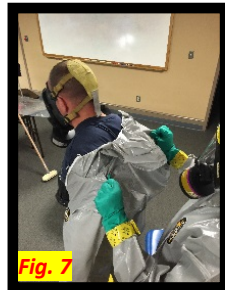


## Step 3: Entrant out of suit continues doffing

- Rotate out and away from the suit (Fig.10)
- Using the gloved hands loosen lower straps of mask (Fig.11)
- Grab mask around the cartridge area, lift out and up to remove (Fig.12) Hand to assistant or set away (Fig.13)

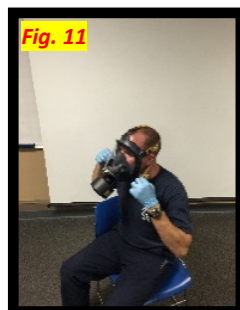

## Step 4: Remove inner gloves turning them inside out, remember don't touch the outside with bare hands (Fig.14,15)

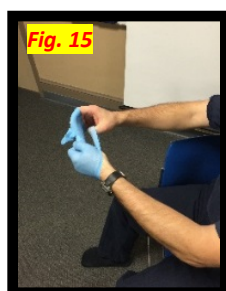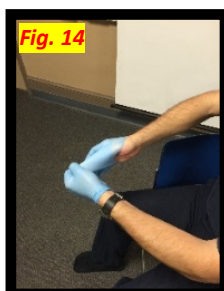

Courtesy of the  
West Hartford  
Fire Department

## Step 5: Wash hands at a minimum, then ideally proceed to shower.